


# QUICK START GUIDE

SE-1515 DP/DX PC-Based ECG Machine

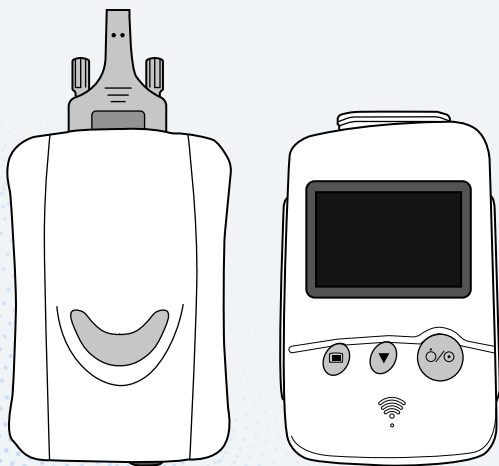



## **SE-1515 ECG Machine**

### **Contact Us:**

info@mdprousa.com  
888-602-7767

# Contents

## 02

---

### Basic Setup

- 02 Downloading the Software on the SE-1515 DP/DX
- 02 Connecting the Cables on the SE-1515 DP
- 03 Connecting the Cables on the SE-1515 DX
- 04 Powering on the SE-1515 DX

## 06

---

### FAQ

- 06 How to Display DOB on ECG Report

# Basic Setup

## Downloading Software For The SE-1515 DP/DX PC-Based ECG Machine


1. Before setting up the SE-1515 DP / SE-1515 DX PC Based ECG Machine, install the software on your PC. The ECG must be plugged into the PC before accessing the software.

**NOTE:** This software is compatible **exclusively with Windows operating systems** and is not supported on Mac.


- If your PC has a CD Drive, use the included CD to download the software.
  - If your PC does not have a CD drive, scan the **QR code on page 7** to download the software.
2. Follow the instructions in the installation guide to install the software.
- 

## Connecting The Cables On The SE-1515 DP

1. Connect the USB Connection Cable to the 6-pin port at the bottom of the sampling box. **(Figure A)**
2. Connect the other end of the USB Connection Cable to the USB port of your PC. Once connected, the LED light on the sampling box will turn on.

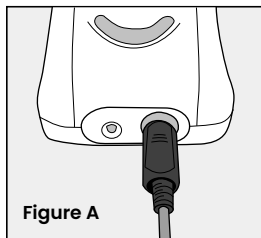

3. Connect the ECG Cable to the Patient Cable Port at top of the sampling box. Secure with two screws. **(Figure B)**


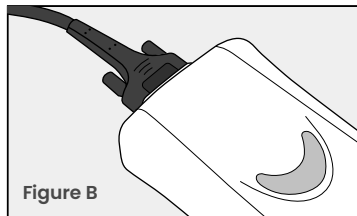


4. The SE-1515 DP is ready to be used with the installed software.

---


## Connecting The Cables On The SE-1515 DX

1. Connect the ECG Cable to the Patient Cable Port at the top of the transmitter. **(Figure C)**
2. Connect the USB Connection Cable to the 6-pin port at the top of the receiver. Then, connect the other end of the USB Connection Cable to the USB port on your PC.
3. Once connected, the green LED on the receiver will turn on. **(Figure D)**

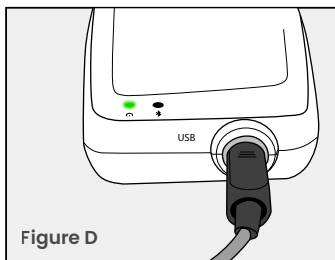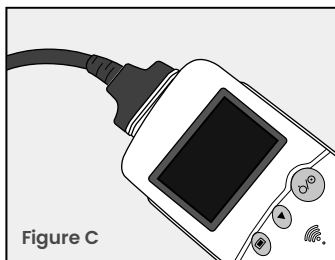

## Powering On The SE-1515 DX

1. Slide the battery compartment door down to remove. Place two AA batteries in the appropriate slots, then replace the battery compartment door. **(Figure E)**

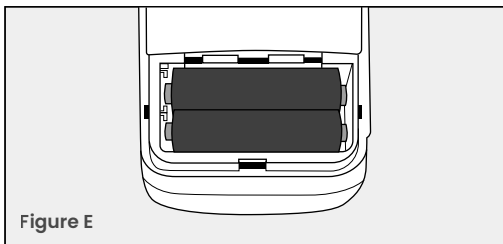

2. Turn on the transmitter by pressing and holding the power button until it powers on.
3. To connect to the receiver, press the 'menu button', then scroll down until you see 'Match Device'.
4. Press the menu button to select. 'Match Device Inquiring' will display for 10 seconds to search for the receiver.
5. Once the receiver address displays on the screen, press the menu button to select.
6. If successful, a Bluetooth icon should be present at the top of the screen.
7. The SE-1515 DX is ready to be used with the installed software.

## **BASIC SETUP COMPLETE**

*Continue to Next Pages for  
FAQ & Troubleshoot Issues*

# FAQ

## How to Display DOB on ECG Report

1. Setting > Date of Birth
2. Reset software
3. Open the software Resting ECG > Enter DOB > OK
4. Run the ECG by pressing 'START'.
5. From the second screen, go to:  
Setting > Print Setting > Normal ECG > Report Template > Edit
6. Then, customize your report to the available selections.
7. Once you customize the report, you can SAVE AS and name the report. Make sure the customized name is displayed on the Current Template.





*Scan here to access  
the download link  
for the software*



*Scan here for additional  
assistance setting up  
your product*



4204 Jutland Drive  
Suite A, San Diego  
CA, 92117

[info@mdprousa.com](mailto:info@mdprousa.com)  
888-602-7767