


# QUICK START GUIDE

SE-301 ECG Machine

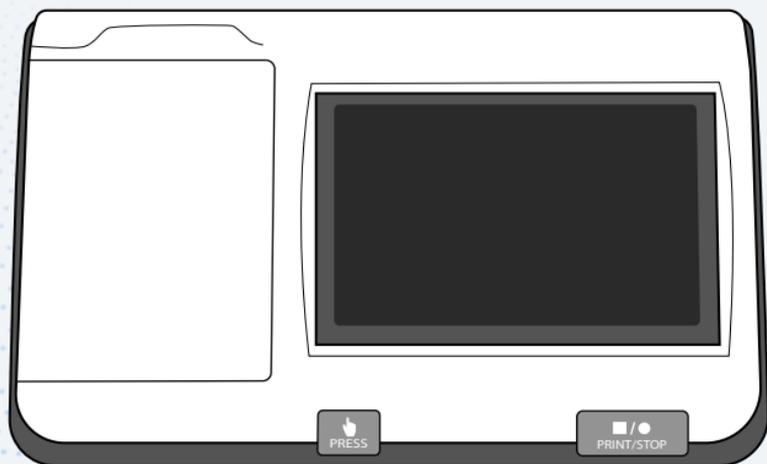



**SE-301 ECG Machine**

**Contact Us:**

[info@mdprousa.com](mailto:info@mdprousa.com)

888-602-7767

# Contents

## 02

---

### **Basic Setup**

- 02 Installing The Battery
- 03 Powering On The SE-301
- 03 Connecting The Patient Cable To The ECG Machine
- 04 Loading Paper Into The Printer
- 05 Settings Change To Print Interpretation

## 07

---



### **FAQ**

- 07 "Lead Off" Error


# Basic Setup

## Installing The Battery

1. Gently flip the SE-301 upside down (screen side down) to reveal the underside of the machine.
2. Use a screwdriver to unscrew the battery compartment cover off. **(Figure A)**
3. Insert the battery wire into the socket. Then, place the battery in the compartment. **(Figure B)**

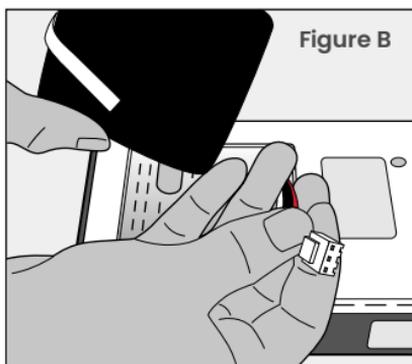
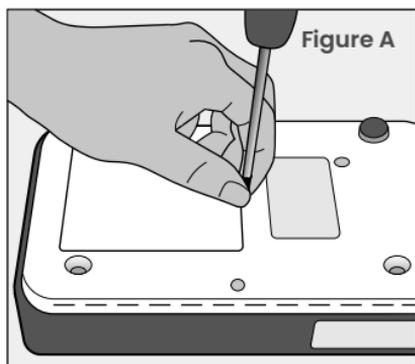


4. Ensure that the white tab is at the top. **(Figure C)**
5. Replace the battery compartment cover, and revert the machine back to its original position.

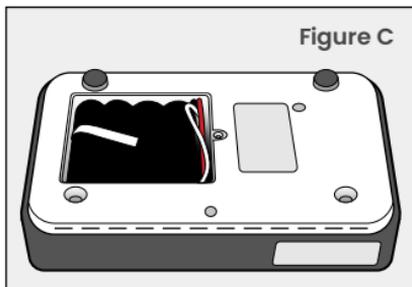

## Powering On The SE-301


1. Plug one end of the AC Power Cable into the power supply socket, located on the backside of the SE-301. Plug the other end into an AC Wall Outlet. **(Figure D)**
2. Press and hold the power button on the right side of the SE-301 to power on the machine. **(Figure E)**

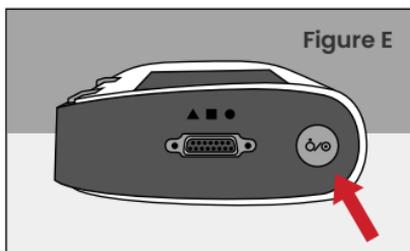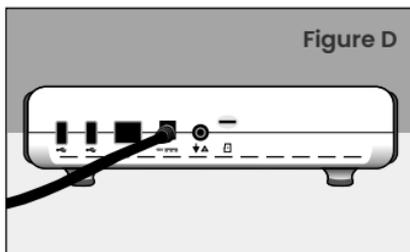

---



## Connecting The Patient Cable To The ECG Machine

1. Connect the patient cable to the patient cable socket on the right side of the main unit, and then secure them with two screws. **(Figure F)**

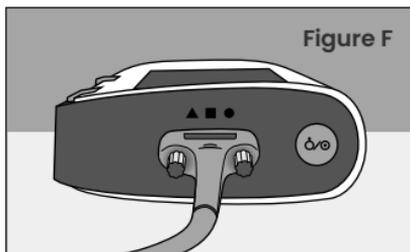


## Loading Paper Into The Printer

1. Push the 'Paper Replacement' button to open the printer door. Lift the door all the way open. **(Figure G)**
2. Put the roller through the paper roll, and insert into the paper compartment. **(Figure H)**
3. Before placing the paper into the printer, orient the paper so that the black square is in the bottom left corner.

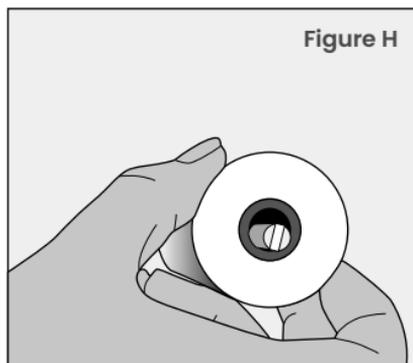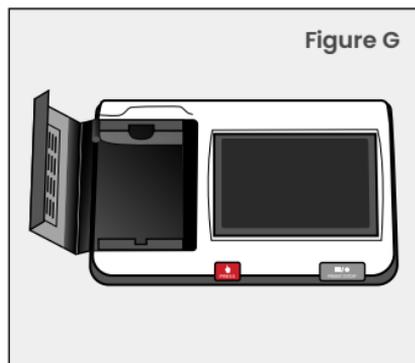

4. Close the printer door, with the first page of paper hanging over the printer. Tear the first page, and the printer is ready to be used. **(Figure I)**

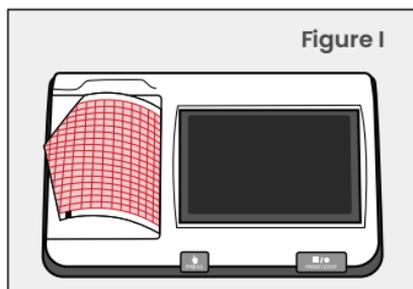

## Settings Change To Print Interpretation

1. In system setup, click 'record info'.
2. Select 'paper marker'.
3. Switch to 'no'.
4. The SE-301 is ready to print interpretations.

BASIC SETUP COMPLETE

*Continue to Next Pages for  
FAQ & Troubleshoot Issues*

# FAQ

## **“Lead Off” Error**

The leads are not connected well. Please check whether the electrodes are connected to the patient skin well, and then make sure that the patient cable socket is connected to the patient cable firmly.



*Scan here for additional  
assistance setting up  
your product*



4204 Jutland Drive  
Suite A, San Diego  
CA, 92117

[info@mdprousa.com](mailto:info@mdprousa.com)  
888-602-7767